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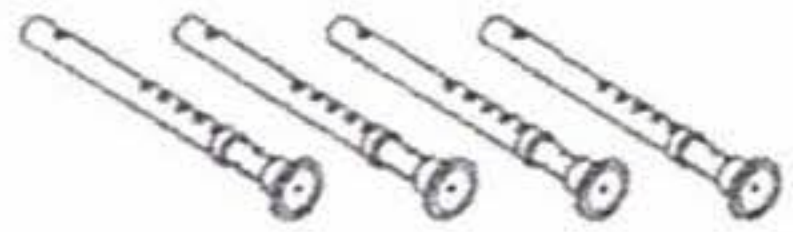
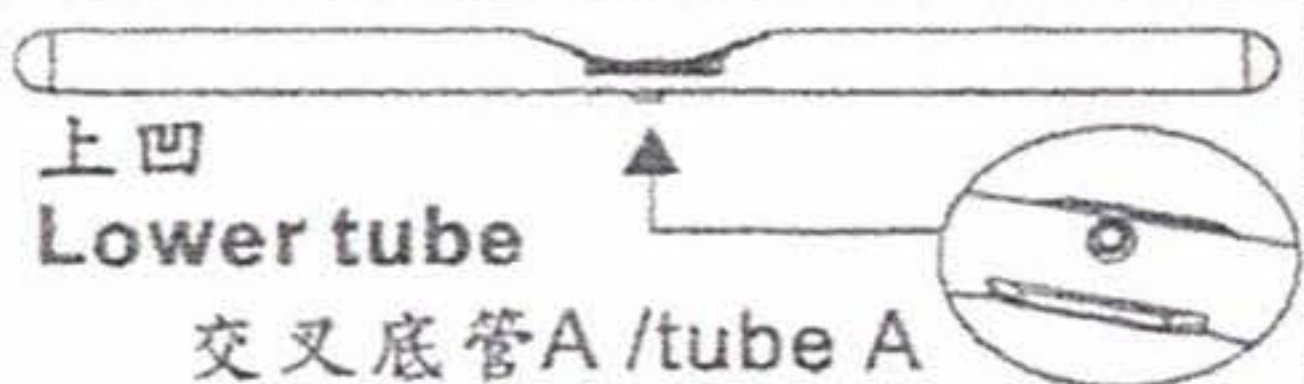
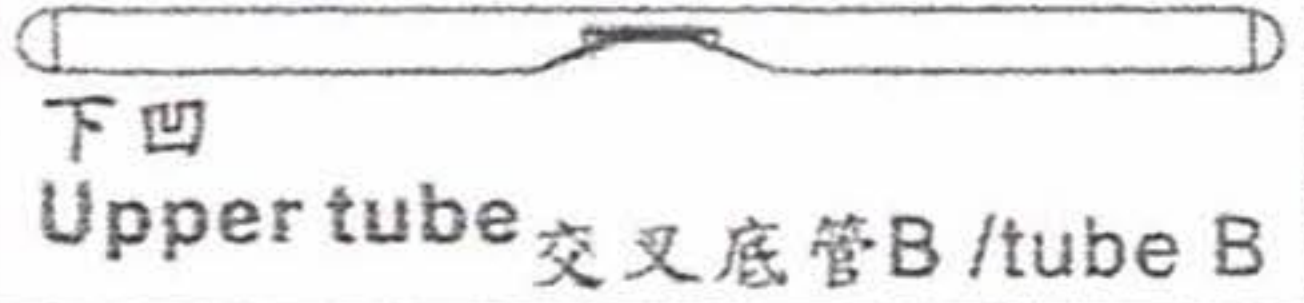
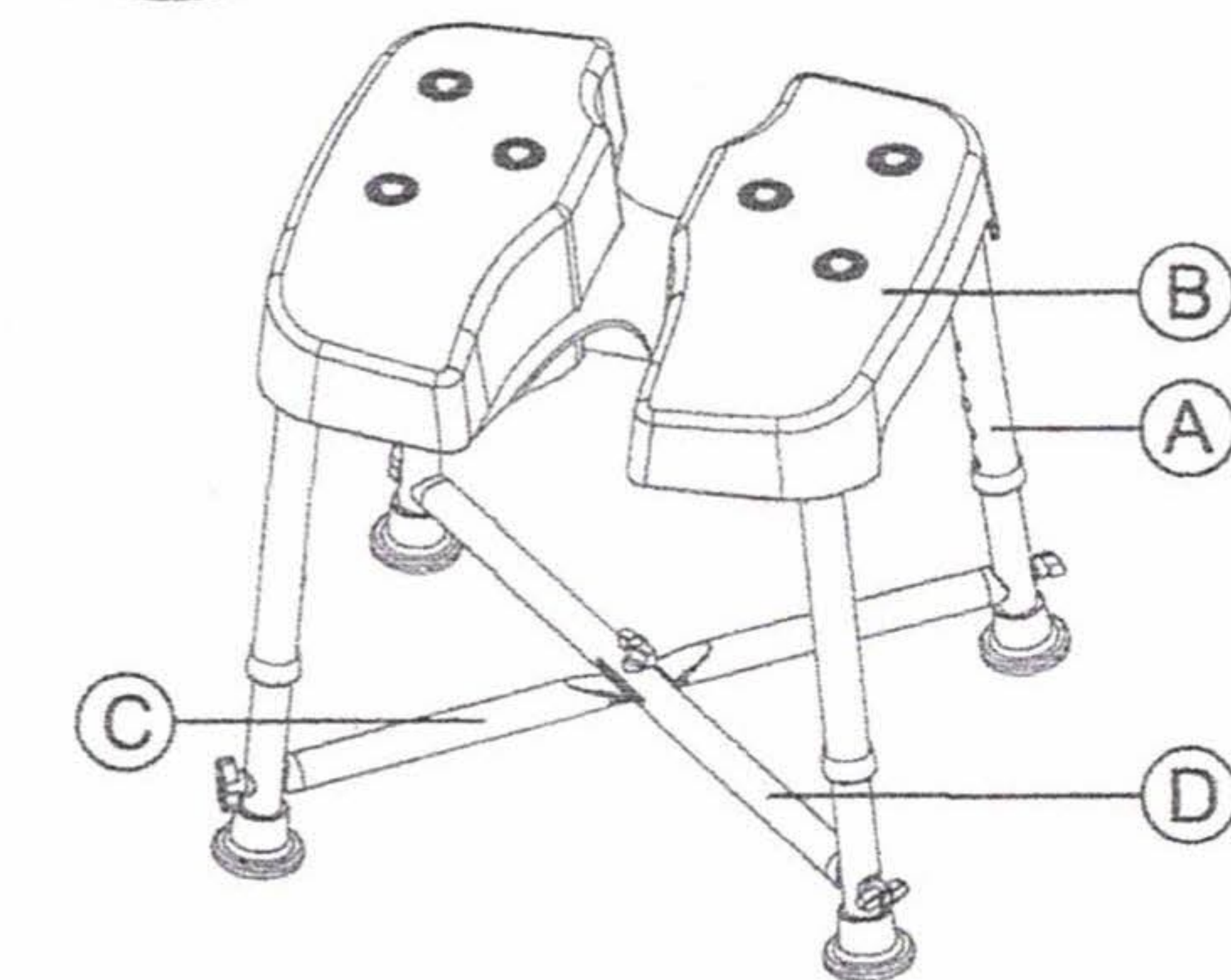
Dura Hygienic Cutout Shower Seat

FDA CE

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

台灣製
Made In Taiwan

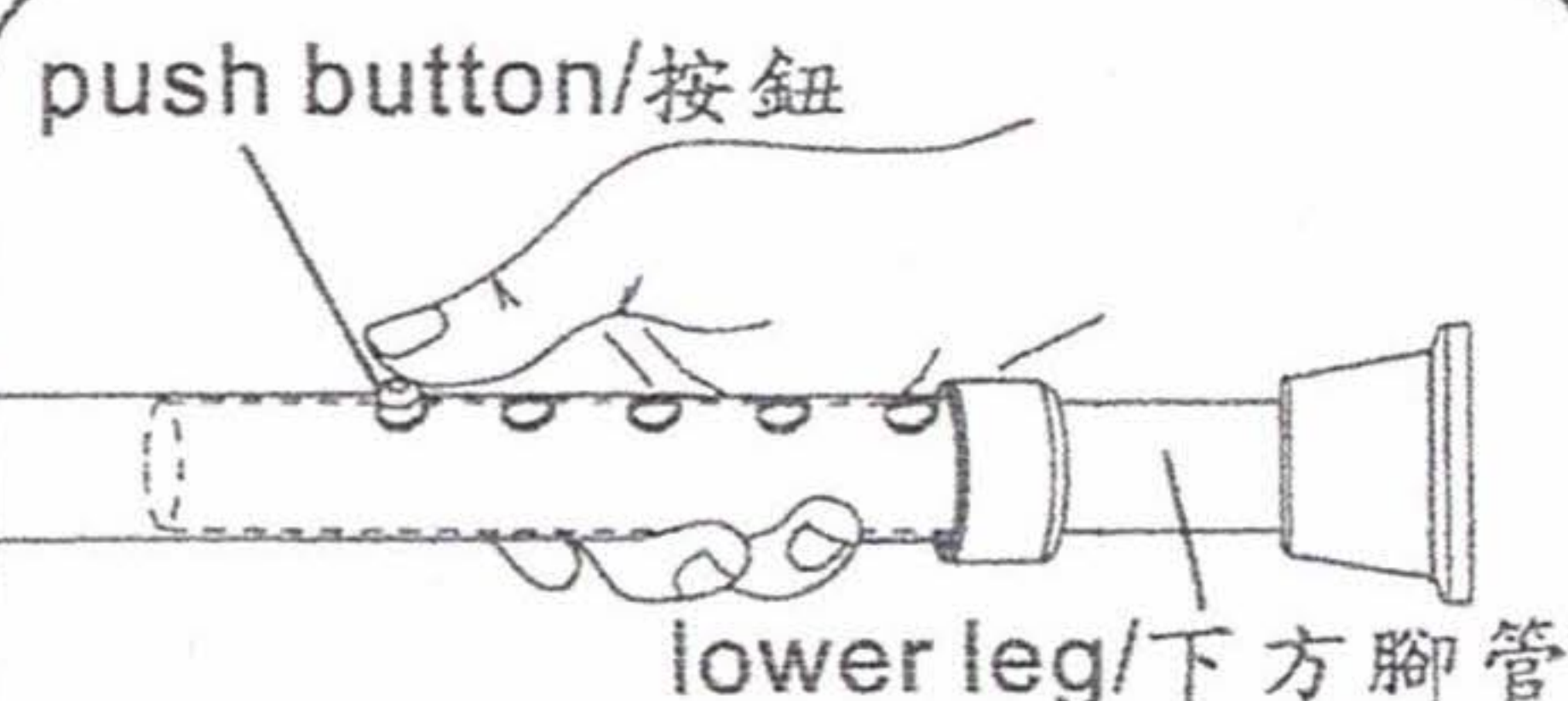HEIGHT ADJUSTMENT 高度調整說明


Depress the push button and pull the lower leg to the desired length; ensure that the push button snaps through the latch hole. Repeat with remaining legs, so that all legs are adjusted to the same length.

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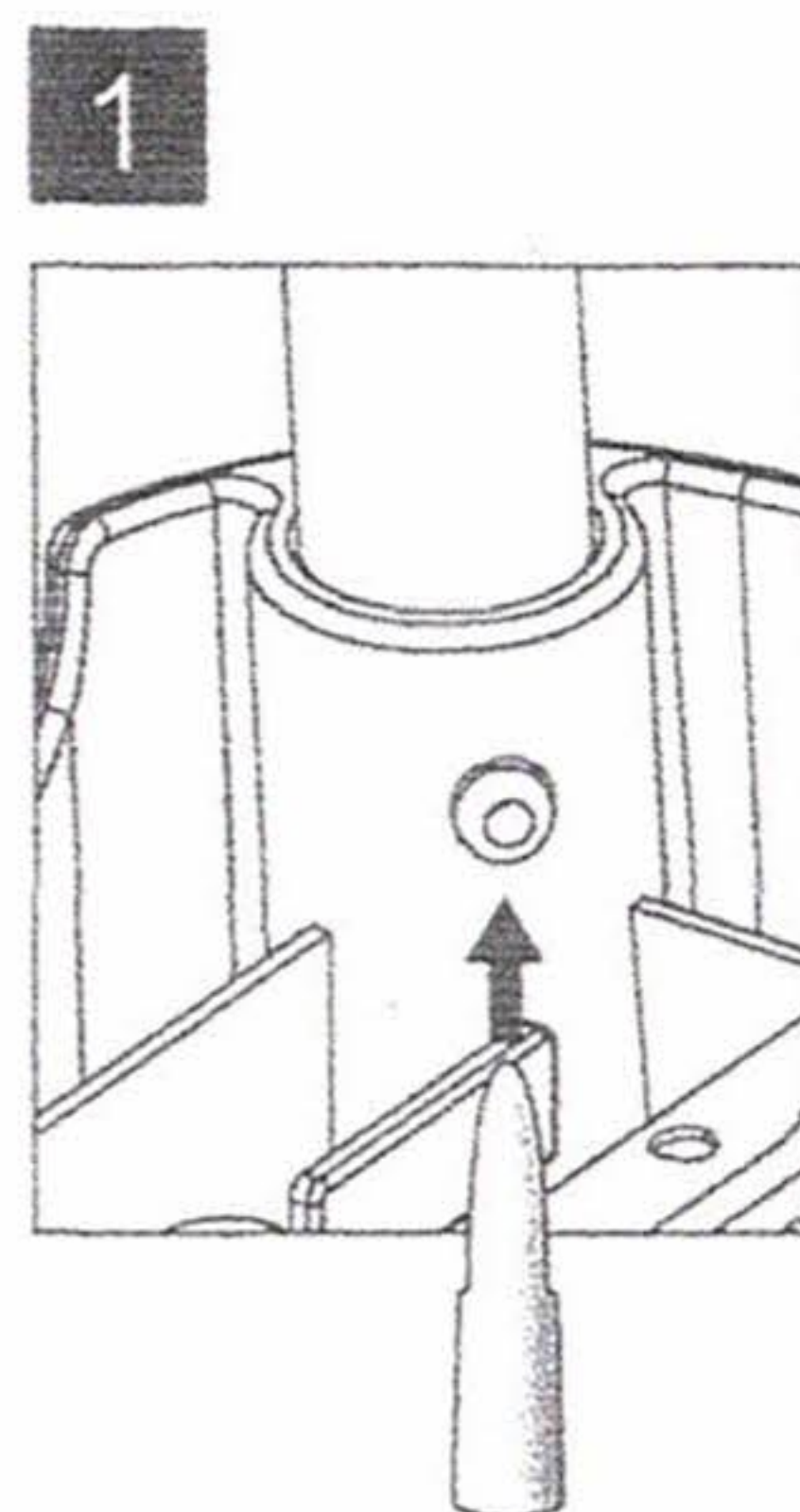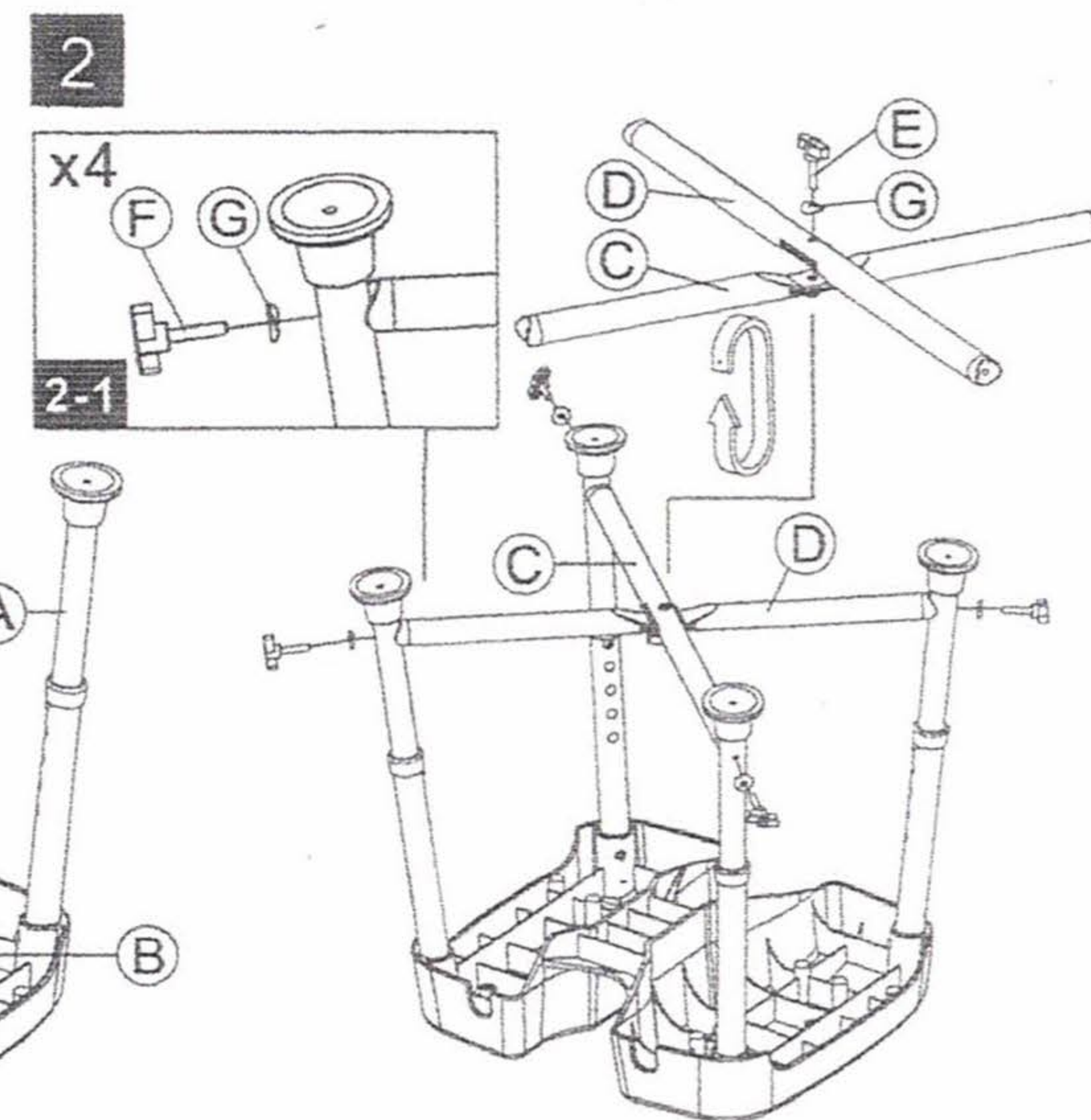Remove leg tubes from the hub with small stuff.
移除腳管使用細小物件插入拔除。

Instructions:

1. Depress the push button and pull the lower leg to the highest height. Insert each leg tube (A) into the seat (B). (Caution: For secure installation, make sure the button on each leg is fully engaged to the hub on the seat).
2. Put tube (D) on (C) in the cross position and then screw tube (D) on (C) with parts (E) & (G) in the center. After assembled, turn the cross tube set to opposite side. Put (C) as the direction remarked (2) in seat, and put (D) as the direction remarked (1) in seat. As shown in partial magnified figure, screw parts (F) & (G) with leg tube tightly.

CAUTION:

1. Read instruction before assembly.
2. Please check before assembling if the parts quantities are correct.
3. Should be checked before each use to make sure it is securely assembled. Reinstall if necessary.
4. Max User Weight (500lbs. / 227kgs)
5. Do not sit alone on the corner of the seat to avoid injured.
6. Loosely install each screw one by one firstly. Once all screws are inserted, tighten all screws fully.
7. To adjust the height, loosen 5 screws inserted in the cross tube set first, and then depress the push buttons and pull the lower legs to the desired height.
8. MedGear assumes no liability for any damage or injury caused by improper assembly or use of this product.

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