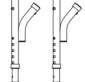
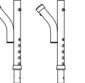

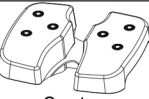

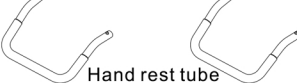
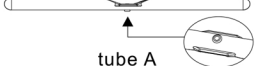
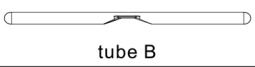





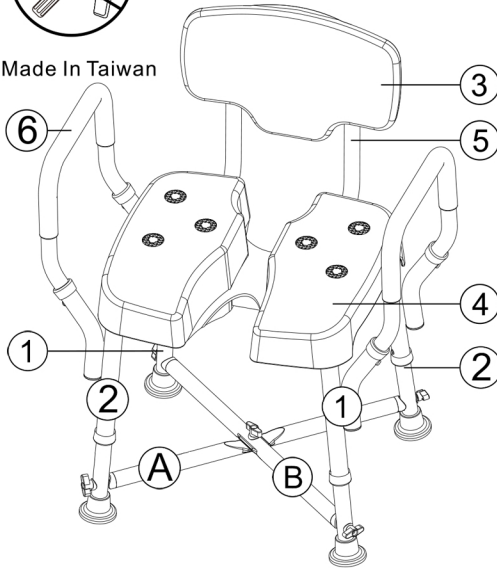
# A-0273A Dura Hygienic Cutout Shower Chair With Armrest

**FDA** **CE**

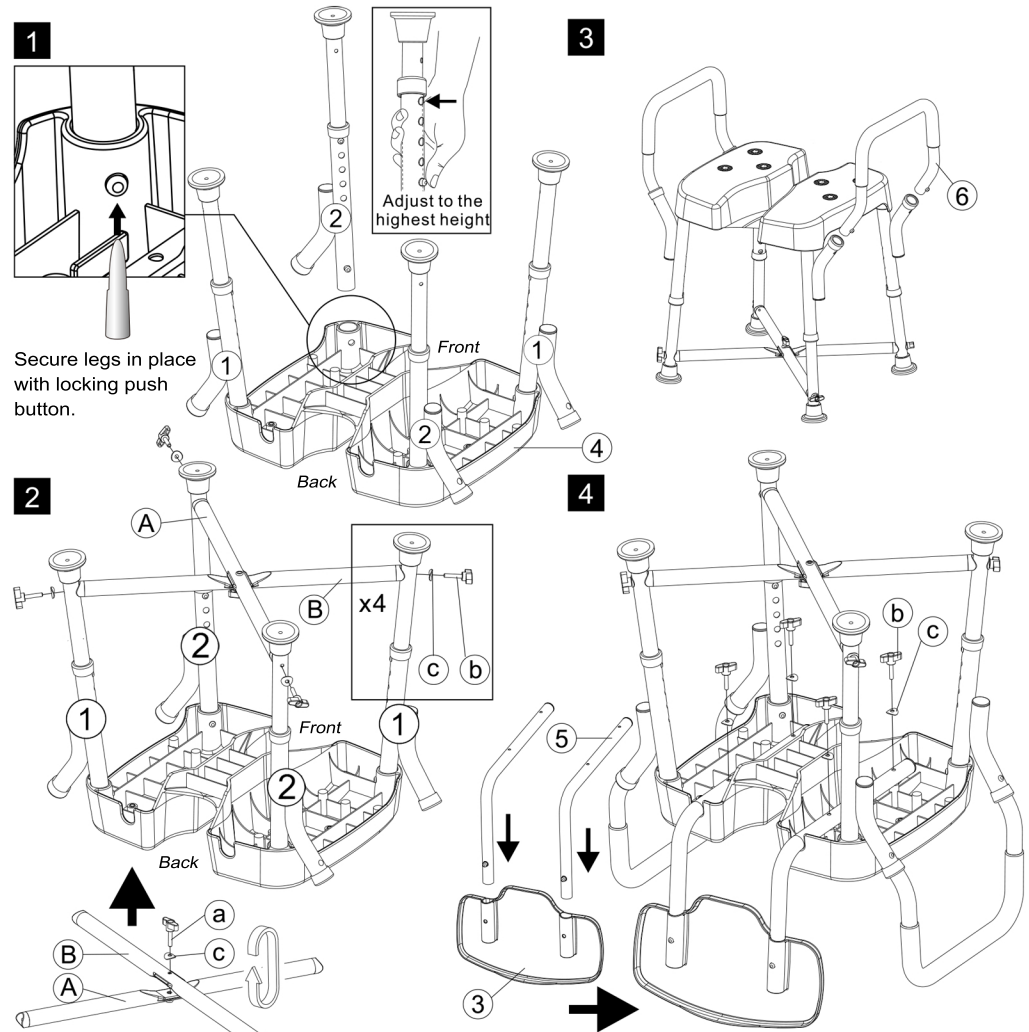
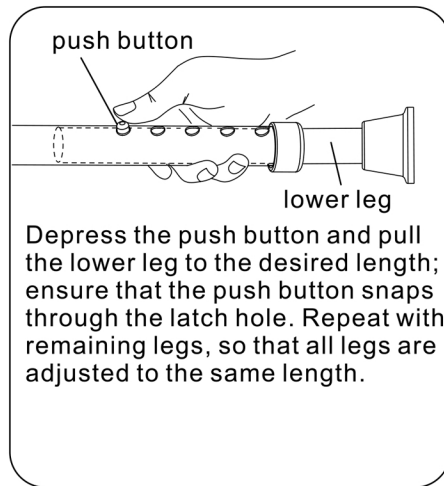
Parts	NO.	Qty
 Leg tube-1	①	2
 Leg tube-2	②	2
 Backrest	③	1
 Seat	④	1
 Tubes for backrest	⑤	2
 Hand rest tube	⑥	2
 tube A	Ⓐ	1
 tube B	Ⓑ	1
 Screw-M6x28L	Ⓐ	1
 Screw-M6x38L	Ⓑ	8
 R Type Washer	Ⓒ	9



Made In Taiwan



## HEIGHT ADJUSTMENT



## Instructions:

1. Insert each leg tube ① into the seat at the Front Left and Back Right positions. Insert leg tube ② into the seat at the Front Right and Back Left positions. (Caution: For secure installation, make sure the button on each leg is fully engaged to the hub on the seat.) Depress the push button and pull the lower leg to the highest height.
2. Put tube (B) on (A) in the cross position and then screw tube (B) on (A) with parts (c) & (a) in the center. Once assembled, turn upside down and assemble (A) with (2), and (B) with (1). As shown in partial magnified figure, screw parts (b) & (c) with leg tube tightly.
3. Assemble hand rests into leg tubes.
4. Assemble the backrest into back tubes. Fix the four points of tubes (5) onto seat with parts (b) & (c). (Caution: For secure installation, make sure the button on each tube is fully engaged to the hub on the backrest).

## CAUTION :

1. Read instruction before assembly.
2. Please check before assembling if the parts quantities are correct.
3. Should be checked before each use to make sure it is securely assembled. Reinstall if necessary.
4. Max User Weight (500lbs. / 227kgs)
5. Do not sit alone on the corner of the seat to avoid injured.
6. Loosely install each screw one by one firstly. Once all screws are inserted, tighten all screws fully.
7. MedGear assumes no liability for any damage or injury caused by improper assembly or use of this product.